

LIST OF BLIFT + PROGRAMS



+33 4 71 65 02 16



sav@electroniquedumazet.com

Mazet Electronics ZA Route de Tence, 43520 le
Mazet-Saint-Voy



Care	Accessories	Modes	Stages	Time / Pressure Level
SCARS	Suction head	Pulsed	Mobilize	5min/ Level 2
	Suction head	Continuous	Smooth	5min/ Level 2
	R35 suction head	Continuous	Vascularize	5min/ Level 1
	Palpate and roll	Continuous	Smooth	10min/ Level 3
DRAINAGE	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Drain	10min/ Level 3
	Suction head	Pulsed	Stimulate	5min/ Level 3
	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Drain	5min/ Level 3
	Suction head	Pulsed	Stimulate	5min/ Level 3
	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Drain	5min/ Level 3
TENDINOPATHY	Suction head	Pulsed	Relieve	5min/ Level 3
	Suction head	Continuous	Mobilize	5min/ Level 3

Care	Accessories	Modes	Stages	Time / Pressure Level
RACHIS	R15 / R25 suction head	Continuous	Vascularize	5min/ Level 3
	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Soften	5min/ Level 3
	Suction head	Continuous	Vascularize	5min/ Level 4
	Suction head	Pulsed	Mobilize	5min/ Level 4
	Suction head	Continuous	Soften	5min/ Level 4
	Suction head	Continuous	Vascularize	5min/ Level 3
	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Soften	5min/ Level 3
	Suction head	Continuous	Vascularize	5min/ Level 3
	Suction head	Pulsed	Mobilize	5min/ Level 3
	Suction head	Continuous	Soften	5min/ Level 3
BURNS	Suction head / Palpate and roll	Pulsed	Mobilize	5min/ Level 2
	Suction head	Continuous	Soften	5min/ Level 3